The natural way of Sweden

SISSEL® FITBAND PLUS & ESSENTIAL

- Gebrauchsanweisung
- Instructions for use
- Mode d'emploi
- Gebruiksaanwijzing
- Instrucciones de uso
- Istruzioni per l'uso

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General information

Read the following user manual carefully and observe the safety instructions and warnings. The SISSEL® Fitband Plus / Essential is a high-quality elastic band for varying and effective support during fitness training, gymnastics and physiotherapy. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the Fit Bands, e.g. in physiotherapy or rehabilitation treatment, should preferably be carried out by or under the guidance of professional users. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may be used in accordance with this operating manual or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the system outside the applications described in these instructions for use. The instructions for use must be kept carefully and within easy reach and must be included if the product is passed on. Ultimately, the regulatory requirements applying to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the integrity of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- ! Do not use the product in the immediate area of heat sources (e.g. radiator, hotplate, open flame, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointed objects.
- ! Practice only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges or anything similar.
- ! Use the product only if you can ensure correct performance of the exercise recommendations. Consult specially trained personnel, physiotherapists, trainers or a medical specialist if you are unsure or have any questions.
- ! Choose an exercise program that fits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.
- ! To prevent injuries and risks of damage during training, do not wear jewelry or long fingernails.
- ! Use by children only under adult supervision and after instruction accompanying therapy. Not suitable for young children.
- ! For bands with clip: Make sure that the clasp is firmly engaged, does not pop open and the band cannot come loose! Make sure that the door remains closed during the exercises.
- ! Do not overstretch the strap and always ensure that accidental slipping of the strap from your hand or other objects (e.g. door) cannot cause a fall or injuries. Always keep the tape away from your eyes.
- ! Only use accessories intended by the manufacturer.
- ! Use by children, adolescents and during pregnancy as well as in case of existing health conditions such as dizziness, cardiovascular problems, joint weaknesses, etc., only under professional supervision and after consulting a doctor.
- ! The product and its accessories are no toys and should be kept out of the reach of children. Do not allow children to play with it. Risk of strangulation and suffocation!

Product information

Scope of delivery: One of the following products.

- 1 x SISSEL® Fitband Plus, 1 x instruction manual (incl. clip system for approx. 7.5 x 2 m tape)
- 1 x SISSEL® Fitband Essential, 1 x instruction manual

Suitable for beginners and advanced users alike, the SISSEL® Fitband Plus / Essential in its high-quality latex version offers a wide range of applications in therapy, rehabilitation, fitness training, strength building, muscle training, body awareness, and improving balance and mobility. The SISSEL® Fitband Plus / Essential is available in 4 color coded intensities (yellow: light, red: medium, green: strong, blue: extra strong). In addition, the SISSEL® Fitband Plus is available with a width of 15 cm in lengths of 5 m, 25 m and 46 m (roll goods) as well as with a width of 7.5 cm in length of 2 m. This allows various exercises and training programs to be adapted to individual goals and abilities of beginners as well as professional athletes.

Technical specifications

Material:	allergen reduced natural latex
Size (W x L):	SISSEL® Fitband Plus: ca. 15 cm x 5 m/25 m/46 m (roll goods) SISSEL® Fitband Plus: ca. 7,5 cm x 2 m SISSEL® Fitband Essential: ca. 15 cm x 2,5 m

Note: Latex products can trigger allergic reactions. Although allergy-causing allergens have been reduced in the SISSEL® Fitband Plus / Essential through a special manufacturing process, there is always a risk of an allergic reaction for the user.

Intended use

- The SISSEL® Fitband Plus / Essential is an elastic band that can be used for therapeutic and sports-related purposes in physiotherapy, rehabilitation, gymnastics and (competitive) sports for varied and effective support.
- The SISSEL® Fitband Plus / Essential should be primarily used by trained professionals. However, after instruction and with some practice, the band can also be used independently.
- For your safety, consult your doctor for an examination if you have any known health restrictions such as cardiovascular diseases, spinal problems or joint pain, or if you are a sports beginner or have not exercised for a long time.
- Warm up before exercising to prepare your body for the workout and to protect yourself from muscle and tendon injuries.
- Perform the training exercises with the SISSEL® Fitband Plus / Essential according to the given instructions.
- Start with lower intensity resistance until you have a good command of the movements. Slowly increase the number of repetitions and the resistance.
- Adapt your training to your personal performance and your training goal. Seek advice for your training plan from qualified personnel if necessary.

- Perform the exercises in a controlled and slow manner. Breathe out during the effort and in again after the effort. Breathe evenly during the exercise, avoid press breathing and never hold your breath.
- When performing the exercises, make sure that the SISSEL® Fitband Plus / Essential is always slightly under tension.
- The number of rounds (sets) and repetitions should correspond to your ability and training goal. It is important to perform the exercises correctly.
- Take sufficiently long breaks between the runs (sets) for regeneration.
- Stop the training immediately if pain occurs.

 Use the product only in accordance with the instructions. Instruction and supervision by qualified personnel is recommended.

Indication

General strength training, strengthening of deep muscles, inter- and intramuscular coordination training, improvement of balance, coordination and joint stability, reduction of postural deficits, extension and optimization of the usual fitness program, preparation and rehabilitation of major operations under medical and/or therapeutic guidance, supervision or instruction, support of many rehabilitation measures.

Contra Indication

Acute pain, severe tension, unstable fractures in the spine, pelvis, upper and lower extremities, with pronounced joint discomfort and pain, acute cardiovascular problems.

Regulatory requirements

The present product of SISSEL® GmbH complies with the following regulatory requirements:

- Regulation (EU) 2017/745 on medical devices

All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessory

The product may only be used with the accessories provided for this purpose.

Usage

1. Check the packaging and the product for any damage upon delivery.
 Never use a damaged product.
2. Check the product for integrity and contamination before each use.
3. Use the product as intended.

Exercises
on pages
26-31

Note: Cleaning measures may be necessary after use (see Care & Hygiene).

Care & Hygiene

- This product is designed for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and fault-free function must be ensured.
- The SISSEL® Fitband Plus / Essential can be cleaned carefully with pure water and a clean cloth if it becomes dirty. Afterwards, hang to dry in the air.
- Due to its special manufacturing process, the SISSEL® Fitband Plus / Essential requires only minimal powdering for care. An additional self-performed powdering is not recommended!
- Do not use disinfectants to clean the SISSEL® Fitband Plus / Essential!
- It is best to store the SISSEL® Fitband Plus / Essential rolled up in a bag or box to protect it from dirt, sunlight and heat.

Disposal & Environment

Product: The product can be disposed of with household waste.

Packaging: No special types of packaging are used. The respective valid disposal guidelines must be observed.

Warranty

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

Note: Operators of this product must observe the regulatory requirements for medical products applicable at the place of use.

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