SISSEL

The natural way

SISSEL® Knee Roller

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SISSEL® Knee Roller

General information

Read the following user manual carefully and observe the safety instructions and warnings. The SISSEL Knee Roller is a multifunctional aid designed to effectively support physiotherapeutic training programs in rehabilitation and prevention, as well as to securely position the knee during treatment. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the product or how to avoid them. The therapeutic use of the SISSEL Knee Roller, e.g. in physiotherapy or rehabilitation treatment, should preferably be carried out by or under the guidance of professional users. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product may only be used in accordance with these instructions for use. SISSEL GmbH accepts no liability for the use of the product outside the applications described in these user instructions. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the condition of the product before each use. If damaged, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate area of heat sources (e.g. radiator, hotplate, open flame, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointed objects.
- ! Exercise only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train exclusively barefoot or wear socks with a non-slip base. Do not train with shoes and make sure you wear suitable sports clothing. If necessary, place a towel over it.

 Risk of injury! Choose a training program that matches your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.

- ! Only use accessories provided by the manufacturer.
- ! Application in children, adolescents, and during pregnancy, as well as in the presence of health issues such as dizziness, heart/circulatory problems, joint weaknesses, etc., only under professional supervision and after medical consultation.

 Risk of injury! The Knee Roller and its accessories are not toys and must be kept out of the reach of children. Do not allow children to play with it.

Product information

Scope of delivery

consisting of: 1 x SISSEL Knee Roller, 1 x instruction for use

Technical data

Dimensions (L x W x H):	approx. 26,5 x 28 x 28 cm
Support height for the back of the knee:	approx. 22 cm
Material:	50% PE, 50% EVA
Max. user weight:	150 kg

Intended purpose

The SISSEL Knee Roller is a multifunctional aid designed to support physiotherapeutic training programs in rehabilitation or in home settings, as well as for prevention and secure positioning of the knee during treatment. The SISSEL Knee Roller is suitable for use at home or in professional physiotherapy and rehabilitation settings for various medical indications related to the knee. Due to the versatile application possibilities of the SISSEL Knee Roller, targeted exercise programs for beginners to professional training concepts can be appropriately complemented and supported. The compact dimensions of the SISSEL Knee Roller allow for practicing in a supine position and standing while maintaining physiological leg axes. The height makes the training intense and effective. The varying lateral elevations guide the knee joint during exercises, even with different knee circumferences. This prevents compensatory movements and slipping of the knee.

 Risk of injury! Use the product only if you can ensure the correct execution of the described exercise recommendations. Instruction and supervision by professional personnel are recommended.

Intended use

Safe, short-term, non-invasive resting and defined positioning of the knee joint in a supine or lateral position in the event of injury or post-operatively, provides pressure-reducing support in the popliteal fossa, relief, stabilisation, pain relief and supports healing processes. During diagnostic, therapeutic or nursing procedures, a securely positioned and well-stabilised knee enhances the effectiveness, efficiency and safety of the treatments (e.g. manual therapy, massage, mobilisation, physical therapies). The SISSEL Knee Roller is a narrow roller with a continuous, ergonomically designed groove in the centre that varies in width. This special design enables anatomically adapted, centred and relieving positioning of the knee joints. This reduces pressure on sensitive soft tissue and vascular structures and effectively minimises lateral slipping of the limb or involuntary evasive movements during positioning and applications.

The product can be used as an aid for positioning in cases of knee problems, as part of early post-operative physiotherapy training programmes and other therapeutic measures, in rehabilitation and in physical therapy. Treatments and measures can be carried out safely on knees of different sizes or swollen knees (thanks to the channel, which varies in width, even over time as the circumference changes). The above-average diameter raises the knee higher and achieves increased knee flexion. This helps to relieve the lumbar spine in the supine position and reduce muscle tension, and intensifies leg muscle training by extending the range of motion. The narrow roll also allows therapeutic exercises to be performed while maintaining the physiological leg axis in the supine and sitting positions.

Indication

- After cruciate ligament rupture (surgically or conservatively treated)
- After knee surgery (e.g., knee endoprosthesis, ligament surgeries, meniscus surgeries)
- Kneecap instability (patellar dislocation) and other kneecap disorders
- Knee osteoarthritis
- Knee instability
- Positioning of the knee for treatments (e.g., mobilizations, massages, frictions)

Contra Indication

- Fractures when exercises in open kinetic chains are not permitted
- Irritation conditions in/around the knee joint (e.g., inflammations, severe swelling, acute trauma)

Regulatory requirements

The Knee Roller by SISSEL GmbH fulfils the regulatory requirements of Regulation (EU) 2017/745 on medical devices.

 All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessory

The product may only be used with the accessories intended for this purpose.

Usage

1. Check the packaging and the product for any damage upon delivery.

 Never use a damaged product.

2. Check the product for integrity and contamination before each use.

3. Use the product as intended.

4. Warm up before training.

5. Exercise according to the workout instructions or follow the guidance of professionals.

Note: Prolonged pressure may cause the foam to develop permanent deformations. Cleaning and, if necessary, disinfection measures may be required after use (see Care & hygiene).

Care & hygiene

This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and error-free operation must be ensured. The product can be wiped with a damp cloth, then dried, and wiped with mild cleaning and disinfecting agents (e.g., 75 % alcohol). Use a soft cloth and no abrasive agents. Do not use agents with phenol content or those containing the substance diethylene glycol monoethyl ether.

 Only use suitable disinfectants that have been tested for effectiveness to carry out disinfection measures.

-  Disinfectants that can damage the materials of the product must be avoided.
-  The application instructions of the disinfectant manufacturer must be followed.

The use of unsuitable preparations or preparations that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee claims.

Expected service life

The expected service life of the SISSEL Knee Roller is five years, starting from the date of delivery to the end user, provided that the conditions specified by the manufacturer have been observed during storage and transport. The actual functional service life of the product depends on its intended use.

Disposal & environment

The product and the accessories supplied with it can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

The warranty is governed by the legal regulations. Any warranty is excluded if goods delivered by SISSEL GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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