*The natural way of Sweden*

# SISSEL® PACK

- | **Gebrauchsanweisung**
- | **Mode d'emploi**
- | **Instructions for use**
- | **Instrucciones de uso**
- | **Istruzioni per l'uso**
- | **Gebruiksaanwijzing**

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## EN

### General information

- Read the following instructions for use carefully and observe the safety instructions and warnings.
- The SISSEL® PACK is a Hot/Cold pack filled with gel for targeted, local heat or cold treatment.
- As a medical device, it is mainly sold to medical professionals or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the conditions to be treated or avoided with the product.
- The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists.
- SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these instructions for use.

- The instructions for use must be kept carefully and within easy reach and must be enclosed if the product is passed on.
- Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

### Safety instructions

- ! Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed. Leaking hot contents can cause skin burns.
- ! Heat the pack in the microwave or in a water bath. Do not heat the pack in an oven or grill.
- ! Do not leave the microwave or oven unattended when heating the pack. Overheating can destroy the pack and lead to skin burns. Follow the microwave manufacturer's or oven manufacturer's operating instructions carefully.
- ! Knead the pack well after heating. This way the heat is evenly distributed in the gel and no heat pockets are created (danger of burning).
- ! Never place the pack directly on the skin. Always use a thin cotton cloth or a cover to avoid damage to the skin (e.g. frostbite, chilblains, burns).
- ! Check the temperature by gently holding the pack against the inside of the wrist.
- ! Remove the pack immediately if you feel uncomfortable with the cold or heat.
- ! Children or people with cognitive impairments should only use the grain pillows under supervision.
- ! Leaking, damaged or deflated packs must not be used. Danger of burns.
- ! The contents of the pack are not suitable for consumption.
- ! Only use accessories intended by the manufacturer.

! The product is not a toy and should be kept out of the reach of children. Do not allow children to play with it.

## Product information

### Scope of delivery

- SISSEL® PACK
- 1 x SISSEL® PACK, 1 x instructions for use
- SISSEL® PACK with cover
- 1 x SISSEL® PACK, 1 x instructions for use, 1 x cover

### Technical specifications

dimensions (L x W): ca. 12 x 25 cm

Material:

- Cover: Composite film polyamide (PA) and polyethylene (PE)
- Filling: water, glycerine, thickener, citric acid, potassium sorbate, sodium benzoate, colorant, bitterant
- Cover (only item „SISSEL® PACK incl. Cover“):  
100 % polyester

### Intended use

The SISSEL® PACK is a Hot/Cold pack for a drug-free, needs-based, individual hot or cold therapy. Warming and cooling treatments use the reactions of the skin, the subcutis, the musculature and deep tissue layers to the respective temperature stimulus. In heat therapy these are primarily blood circulation-enhancing and metabolism-stimulating effects, in cold therapy heat withdrawal and pain relief. Thus, the grain pillow is an ideal way to use heat and cold in a targeted therapeutic way on many different occasions.

### Application code

Place the pack in the refrigerator or freezer for at least 2 hours (-15°C – 18°C). If necessary, dry off the pack and put it in the corresponding cover or wrap the pack in a thin cotton cloth. Place the pack on the affected area. Only apply the pack for as long as you find it comfortable

– max. 20 minutes. Before applying the pack again, the treated area should remain uncovered for 20 minutes.

### Application heat

Heat the pack in the microwave or in a water bath.

**Water bath:** Bring enough water to a boil to completely cover the pack. Remove the water from the heat source and immerse the gel pack in the hot water for 4 to 6 minutes (max. 80°C). Carefully remove the pack and knead it well to distribute the heat evenly. Dry the compress.

**Microwave:** Make sure that the plate with the pack can rotate freely in the microwave and that there is no food residue in the microwave. Place the pack flat in the microwave and distribute the gel evenly. The actual performance of different microwave brands varies despite the same power specifications. We therefore recommend that you try to find the optimum heating time. You can use the following information as a guide:

300 watts: max. 1 minute, if necessary reheat in 30-second intervals

600 watts: max. 30 seconds, if necessary reheat in 10-second intervals

800 watts: max. 15 seconds, if necessary reheat in 10-second intervals

**Risk of injury!** The heat may be unevenly distributed at first. It is important to knead the pack well after each heating cycle to distribute the temperature evenly in the gel.

**Risk of injury!** Never place the pack directly on the skin, always use a thin cotton cloth or the corresponding cover.

### Indication heat

Tension, sore muscles, myogelosis/hard tension, back pain, tendopathies, muscle and tendon shortening, lumbago, menstrual cramps, myalgias, irritations of the gastrointestinal tract, hypertension (high blood pressure), general nervousness and restlessness, colds.

## Indication cold

Local tissue irritation (swelling, bruising), sports injuries, pain relief, traumatic and rheumatoid arthritis, haematomas (bruises), sprains, scalds or burns, insect bites, headaches, migraines, nosebleeds, fever, tired and heavy legs.

## Contra indication heat

Acute inflammation, acute trauma (accidents, open wounds), acute joint inflammation, oedema, febrile infections, dermatitis, deep vein thrombosis, peripheral arterial occlusive disease, hypotension, heart disease, impaired skin sensation (e.g. diabetics, persons insensitive to heat/cold, children under 4 years). If in doubt, seek medical advice.

## Contra indication cold

Cold intolerance and cold allergy, Raynaud's syndrome, circulatory disorders, acute trauma (accidents, open wounds), peripheral arterial occlusive disease, deep vein thrombosis, impaired skin sensation (e.g. diabetics, individuals) who are insensitive to heat/cold, children under 4 years of age. If in doubt, seek medical advice.

## Regulatory requirements

The following regulatory requirements are fulfilled by this product of SISSEL® GmbH:

- Regulation (EU) 2017/745 on medical devices
- Tested for biocompatibility according to DIN EN ISO 10993-5,-10

All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

## Accessory

The product must only be used with the accessories provided.

## Application

1. Check the packaging and the product for any damage on delivery. Never use a damaged product.
2. Check the product for damage and contamination before each use.
3. Never put the pack directly on the skin, always use a thin cotton cloth or the corresponding cover.
4. Use the product according to above instructions.

## Care & Hygiene

- This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be followed to ensure fault-free functioning.
- If dirty, the pack can be wiped off with a soft cloth and lukewarm water.
- Disinfectants, abrasive cleaners, glass cleaners or all-purpose cleaning agents can damage the surface materials of the product and must not be used.

## Disposal & Environment

- The product and the accessories supplied can be disposed of with household waste.
- No special types of packaging are used. The respective valid disposal guidelines must be observed.

## Warranty

The warranty is governed by the statutory provisions  
**Note:** Operators of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

Any warranty is excluded if goods delivered by SISSEL® GmbH have been handled, treated or modified by others without consent or if the instructions for use have not been followed.

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- Dit product van SISSEL® GmbH voldoet aan de volgende regelgevende vereisten:
SISSEL® GmbH • Bruchstrasse 48 • 67098 Bad Dürkheim • Germany