The natural way

SISSEL[®] PALEA & PANICORE

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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Trimman-Symbol

General information

Read the following instructions for use carefully and observe the safety instructions and warnings. The SISSEL® Palea / Panicore pillow adapts exactly to the shape of the head and neck and provides optimum support for the cervical spine. Its temperature-regulating and moisture-wicking properties promote restful sleep. The SISSEL® Palea / Panicore cushion has been developed to be low-noise and low-dust so as not to disturb sleep. The spelt chaff in the SISSEL® Palea pillow consists of over 90 % silica, which contributes to restful and regenerating sleep thanks to its natural properties. The product may only be used in accordance with these instructions for use. SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these user instructions. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on.

Safety instructions

- ! Check the condition of the product before each use. If damaged, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate area of heat sources (e.g. radiator, hotplate, open flame, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointed objects.
- ! Use only accessories recommended by the manufacturer.

 In case of uncertainty or existing health problems in the shoulder neck area or cervical spine, use the product only after consulting a doctor.

Product information

Scope of delivery consisting of one of the following products:

- 1 x SISSEL® Palea, 1 x instructions for use
- 1 x SISSEL® Panicore, 1 x instructions for use

Technical data

SISSEL® Palea

Dimensions (L x W x H):	approx. 60 x 40 x 6 cm
Material Filling:	100 % spelt
Material Cover:	100 % cotton
Weight:	approx. 1,25 kg

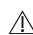
SISSEL® Panicore

Dimensions (L x W x H):	approx. 60 x 40 x 6 cm
Material Filling:	100 % millet chaff
Material Cover:	100 % cotton
Weight:	ca. 1,34 kg

Accessory

The product may only be used with the accessories supplied.

Usage

1. Check the packaging and the product for any damage upon delivery.
-  Never use a damaged product.
2. Check the product for integrity and contamination before each use.
3. Use the product as intended.
4. Use the SISSEL® Palea / Panicore instead of your usual pillow for sleeping.

Note: Cleaning measures may be necessary after use (see Care & hygiene).

Care & hygiene

The SISSEL® Palea / Panicore cushion is not washable!

-  The cushion must be protected from moisture to prevent mold growth.
-  If black stains form, the SISSEL® Palea / Panicore cushion must be disposed of immediately. There is a risk of mold growth.

Note: The SISSEL® Palea / Panicore cannot be disinfected.

Storage: For storage, protect the product from direct sunlight and store it in a clean, dark and dry place when not in use.

Disposal & environment

The product and the accessories supplied with it can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

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