SPINEFITTER

BACK IN BALANCE

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OCS SPINEFITTER
BACK IN BALANCE

1. GENERAL INFORMATION

Read the following instructions for use carefully and observe the safety instructions and warnings.

The SPINEFITTER by SISSEL® Linum / Linum Pro is a Hot/Cold pack filled with linseed for targeted, local heat or cold treatment of the

muscles in the spinal column. Thanks to its disinfectable surface, the Linum Pro can be used multiple times by different users. This makes the Linum Pro ideal for everyday use in physiotherapy and rehabilitation, among other things.

As a medical device, it is mainly sold to medical professionals or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the conditions to be treated or avoided with the product.

The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists.

SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these instructions for use.

The instructions for use must be kept carefully and within easy reach and must be enclosed if the product is passed on.

Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

- ! Check the temperature by gently holding the grain pillow against the inside of the wrist.
- ! Remove the grain pillow immediately if you feel uncomfortable with the cold or heat.
- ! Children or people with cognitive impairments should only use the grain pillows under supervision.
- ! Leaking, damaged or deflated grain pads must not be used. Danger of burns / Choking hazard!
- ! The contents of the grain pillow are not suitable for consumption.
- ! Only use accessories intended by the manufacturer.
- ! The product is not a toy and should be kept out of the reach of children. Do not allow children to play with it.

2. PRECAUTIONARY INSTRUCTIONS

- ! Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed. Leaking hot contents can cause skin burns.
- ! Heat the grain pillow in the microwave or in the oven. Do not heat the grain pillow in a water bath or grill.
- ! Keep the area of use free from sharp or pointed objects.
- ! Do not leave the microwave or oven unattended when heating the cushion. Overheating can destroy the cushion, also there is a risk of fire. Follow the microwave manufacturer's or oven manufacturer's operating instructions carefully.
- ! After heating, shake the grain pillow well. This way the heat is evenly distributed in the filling and no heat pockets are created (danger of burning).
- ! Never place the grain pillow directly on the skin. Always use a thin cotton cloth or your clothing as an intermediate layer to avoid damage to the skin (e.g. frostbite, chilblains, burns).

3. PRODUCT INFORMATION

Scope of delivery

1 x SPINEFITTER by SISSEL® Linum / Linum Pro, 1 x Instruction Manual

Technical specifications

Dimensions (LxWxH): 84 x 12 x 2 cm

Weight: approx. 1 kg

Material:

Cover Linum: 100 % Cotton

Cover Linum Pro: 100 % Polyester with Polyurethane Coating

Filling: 100 % Linseed

Max. user weight: 150 kg

Intended use

The SPINEFITTER by SISSEL® Linum / Linum Pro is a Hot/Cold pack with natural filling for a drug-free, needs-based, individual hot or cold therapy. Warming and cooling treatments use the reactions of the skin, the subcutis, the musculature and deep tissue layers to the respective temperature stimulus. In heat therapy these are primarily blood circulation-enhancing and metabolism-stimulating effects, in cold therapy heat withdrawal and

pain relief. Thus, the grain pillow is an ideal way to use heat and cold in a targeted therapeutic way on many different occasions.

The SPINEFITTER by SISSEL® Linum / Linum Pro is particularly suitable for use on the muscles near the spine due to its elongated shape. It can be used individually or in combination with the SPINEFITTER by SISSEL®, on which it can be securely attached thanks to the side pockets and the central band.

Application of heat

- Only heat the cushion in the microwave or oven.
- **Microwave:** Make sure that the plate with the grain pillow can rotate freely in the microwave and that there is no food residue in the microwave. Place the grain pillow folded in the microwave and distribute the filling evenly if necessary. The actual performance of different microwave brands varies despite the same power specifications. We therefore recommend that you try to find the optimum heating time. You can use the following information as a guide:

600 Watt: 2 minutes

1000 Watt: 60-90 seconds

⚠ Fire hazard! Make sure that the plate can rotate freely in the microwave!

- Remove the grain pillow from the microwave and check the temperature. If further heating is required, shake the product and heat it again for 20 seconds. Then check the temperature again and heat further if necessary until the desired temperature is reached.
- **Oven:** Preheat the oven to 70 °C (top and bottom heat or convection). Place the grain pillow folded flat on a clean rack, if necessary with baking paper and place in the oven for 4 to 6 minutes on the middle shelf.

- Carefully remove the grain pillow from the oven and check the temperature. If further heating is desired, shake the product and heat it again for 1 to 2 minutes. Then check the temperature again and heat further if necessary until the desired temperature is reached.
- Place the grain pillow on the affected area with a thin cotton cloth or your clothing as an intermediate layer. Optional: Attach the grain cushion to the SPINEFITTER as shown and lie down on it. Do not use the grain cushion in combination with the SPINEFITTER for exercises performed in a standing or kneeling position, as slipping of the grain cushion could lead to loss of balance or slipping off the exercise equipment.

⚠ Risk of injury! The heat may be unevenly distributed at first. It is important to shake the cushion well after each heating cycle to distribute the temperature evenly in the filling.

⚠ Risk of injury! Never place the grain pillow directly on the skin, always use a thin cotton cloth or your clothing as an intermediate layer.

Application cold

- Place the grain pillow in a suitable plastic bag in the refrigerator or freezer for at least 2 hours. Place the grain pillow on the affected area with a thin cotton cloth or your clothing as an intermediate layer. Only apply the grain pillow for as long as you find it comfortable - max. 20 minutes. Before applying the grain pillow again, the treated area should remain uncovered for 20 minutes.

⚠ Risk of injury! Never place the grain pillow directly on the skin, always use a thin cotton cloth or your clothing as an intermediate layer.

Indication heat

Tension, sore muscles, myogelosis/hard tension, back pain, tendopathies, muscle and tendon shortening, lumbago, menstrual cramps, myalgias, irritations of the gastrointestinal tract, hypertension (high blood pressure), general nervousness and restlessness, colds.

Indication cold

Local tissue irritation (swelling, bruising), sports injuries, pain relief, traumatic and rheumatoid arthritis, haematomas (bruises), sprains, scalds or burns, insect bites, headaches, migraines, nosebleeds, fever, tired and heavy legs.

Contra indication heat

Acute inflammation, acute trauma (accidents, open wounds), acute joint inflammation, oedema, febrile infections, dermatitis, deep vein thrombosis, peripheral arterial occlusive disease, hypotension, heart disease, impaired skin sensation (e.g. diabetics, persons insensitive to heat/cold, children under 3 years). If in doubt, seek medical advice.

Contra indication Cold

Cold intolerance and cold allergy, Raynaud's syndrome, circulatory disorders, acute trauma (accidents, open wounds), peripheral arterial occlusive disease, deep vein thrombosis, impaired skin sensation (e.g. diabetics, individuals) who are insensitive to heat/cold, children under 3 years of age. If in doubt, seek medical advice.

Regulatory requirements

The following regulatory requirements are fulfilled by this product of SISSEL® GmbH:

- Regulation (EU) 2017/745
on medical devices

Accessories

The product must only be used with the accessories provided.

4. APPLICATION

1. Check the packaging and the product for any damage on delivery.

 Never use a damaged product.

2. Check the product for damage and contamination before each use.
3. Never put the product directly on the skin, always use a thin cotton cloth or your clothes as an intermediate layer.
4. Use the product as intended.
5. Optional: Attach the product to the SPINEFITTER by SISSEL with the sewn-on pockets.

Note: Revised design, suitable for SPINEFITTER by SISSEL® and SPINEFITTER by SISSEL® Extension Kit.

See last page for attachment instructions.

5. CARE & HYGIENE

SPINEFITTER by SISSEL® Linum

- This product is suitable for repeated use. However, personal application is recommended, as classical treatment methods are not applicable to Hot/Cold packs with natural filling and cotton covers.
- The product cannot be washed, cleaned or disinfected. The product must always be protected from moisture.
- Detergents, cleaning agents and disinfectants must not be used.
- For storage, protect the product from direct sunlight and store it in a dark, dry place when not in use.

SPINEFITTER by SISSEL® Linum Pro

- The SPINEFITTER by SISSEL® Linum Pro is suitable for repeated use on additional persons. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and error-free function must be ensured.
- The product can be wiped with standard cleaning and disinfecting agents for polyurethane surfaces.
- Use a soft cloth and no abrasive agents. Do not use agents containing phenol or those containing the substance diethylene glycol monoethyl ether.

- ⚠ Disinfection measures are to be carried out exclusively with suitable disinfectants that have been tested for effectiveness.
- ⚠ Disinfectants that can damage the materials of the product must be avoided.
- ⚠ The application instructions of the disinfectant manufacturer must be observed.

Note: The use of unsuitable preparations or preparations that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee claims.

6. DISPOSAL & ENVIRONMENT

Product: The product and the accessories supplied can be disposed of with household waste.

Packaging: No special types of packaging are used. The respective valid disposal guidelines must be observed.

7. WARRANTY

The warranty is governed by the statutory provisions.

Note: Operators of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

Any warranty is excluded if goods delivered by SISSEL® GmbH have been handled, treated or modified by others without consent or if the instructions for use have not been followed.

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SISSEL® GmbH · Bruchstrasse 48