SPINEFITTER

BACK IN BALANCE

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1. GENERAL INFORMATION

Read the following instructions for use carefully and observe the safety instructions and warnings. The SPINEFITTER by SISSEL Multi Extension Kit is used to extend the SPINEFITTER by SISSEL by 2 pairs of balls using a simple, secure click system. This makes the set ideal for tall users over 180 cm. The product can also be used without the Connector as a stand-alone device to support training, improve spinal and joint mobility, relieve tension, strengthen deep muscles and for balance exercises. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the SPINEFITTER by SISSEL Multi Extension Kits, e.g. in physiotherapy or rehabilitation treatment, should preferably be carried out by or under the guidance of professional users. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists. SISSEL GmbH accepts no liability for the use of the product outside the applications described in these instructions for use. The instructions for use must be kept carefully and within easy reach and must be enclosed if the product is passed on. Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

2. SAFETY INSTRUCTIONS

- ! Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- ! Do not use the SPINEFITTER by SISSEL Multi Extension Kit in the immediate vicinity of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the exercise area free from sharp and pointed objects.
- ! Exercise only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train exclusively barefoot or wear socks with a non-slip base. Do not train with shoes and make sure you wear suitable sports clothing.
- ! Choose a training programme that suits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.
- ! Do not insert fingers or other body parts into the openings of the training device.
- ! Do not bend the individual parts of the SPINEFITTER by SISSEL Multi Extension Kit. Only use and store the Multi Extension Kit on a flat surface and do not place any objects on top of it.

- ! Only use accessories provided by the manufacturer.
- ! Only use the SPINEFITTER by SISSEL Mini on backrests as described. Do not exceed the maximum application time.

- ⚠ Risk of injury! The Carry Strap is not a training accessory and must be removed before use. The Carry Strap is used exclusively for transporting or attaching the SPINEFITTER by SISSEL Mini to a backrest.
- ⚠ Risk of injury! The SPINEFITTER by SISSEL Mini and its accessories are not toys and must be kept out of the reach of children. Do not allow children to play with it.

3. PRODUCT INFORMATION

Scope of delivery

SPINEFITTER by SISSEL Multi Extension Kit: 1 x SPINEFITTER by SISSEL Mini, 1 x Carry Strap, 1 x SPINEFITTER by SISSEL Connector, 1 x instruction manual

Technical data

Dimensions (L x W x H):	Mini: approx. 12 x 12 x 6 cm Connector: approx. 18 x 5 x 4 cm Carry Strap: approx. 170 x 2 cm
Material:	Mini: Polyurethane Connector: Polyurethane Carry Strap: Strap: Polyester, Buckle: Polyamide
Max. User weight:	Mini: 150 kg Connector: 150 kg

Note: The product is made from a high-quality polyurethane foam. Please note that this naturally has a slightly structured, irregular surface. This does not represent a product defect and has no influence on the use or cleaning of the product.

Intended use

Use for general training

The SPINEFITTER by SISSEL Multi Extension Kit in combination with the SPINEFITTER by SISSEL for tall users over 180 cm is an innovative training device that combines relaxation and effective training. Can be used in private or professionally guided individual or group training, the combination or the SPINEFITTER by SISSEL Mini alone is suitable for a variety of exercises thanks to its unique shape and is an ideal support for releasing tension and adhesions and improving spinal and joint mobility. By simply placing the body intuitively on the SPINEFITTER by SISSEL Mini, a loosening, e.g. of the shoulder girdle, can already take place and a stimulation of the energy points along the spine can take place. This makes it the perfect everyday partner to bring the body into balance and promote mobility.

Therapeutic use

The SPINEFITTER by SISSEL Multi Extension Kit, in combination with the SPINEFITTER by SISSEL for large users, or the SPINEFITTER by SISSEL Mini alone are used as a portable therapeutic aid for active loosening of myofascial tissue, loosening and relaxation of muscles close to the spine, individual massage, stimulation of blood circulation and metabolism as well as mobilisation of the spine in the area of the vertebral bodies. In addition, the product can be used as part of physiotherapeutic therapy programmes and as a therapeutic aid in the standing position for functional balance exercises, psychomotor training, for training body awareness and for fall prevention exercises.

Use as a back support

Thanks to its unique design, the SPINEFITTER by SISSEL Mini attached to a backrest allows tension to be released by detonating the muscles and mobilizing the spine into an upright position. It is also possible to add the SPINEFITTER by SISSEL Trigger Tool to intensify trigger point treatment.

Indication

- Chronic shoulder-neck tension
- Neck tension and neck pain
- Tense, hardened muscles near the spine
- Movement restrictions of the cervical spine, thoracic spine, shoulder joints
- Diffuse pain in the shoulder, neck, spine, lumbar spine
- Cervical, thoracic and lumbar spine syndrome
- Therapeutic aid for physiotherapy, osteopathy, sports medicine, rehabilitation
- Supporting aid for measures to improve posture, body awareness, body perception, movement sequences, balance, general coordination, psychomotoric skills, coordination
- Addition to and optimisation of the usual fitness programme

Contra indication

- Lack of compliance (e.g. due to fear or pain).
- Acute injuries (open wounds, soft tissue injuries, unstable fractures in the spine, pelvis, extremities, etc.)
- Acute radiating pain, paraesthesia, movement disorders (consult specialist or physiotherapist)
- Spondylodesis
- Osteoporosis
- Intervertebral disc problems
- Acute movement disorders
- Pronounced sensory perception disorders
- Usage by children and adolescents, during pregnancy or in case of general health problems only under professional supervision and after professional consultation

- ⚠ Risk of injury! Use the SPINEFITTER by SISSEL with the Multi Extension Kit in case of pronounced sensory perception disorders, unstable fractures, open wounds, acute injuries and movement disorders only after prior consultation with a doctor.
- ⚠ Risk of injury! Only use the SPINEFITTER by SISSEL with the Multi Extension Kit if you can ensure correct performance of the exercise recommendations described. Instruction and supervision by qualified personnel is recommended.

Regulatory requirements

The SPINEFITTER by SISSEL Multi Extension Kit by SISSEL GmbH fulfils the regulatory requirements of Regulation (EU) 2017 / 745 on medical devices.

- ⚠ All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessories

The SPINEFITTER by SISSEL Multi Extension Kit may only be used with accessories offered or approved by the manufacturer (see spinefitter.com).

4. USAGE

1. Check the packaging and the product for any damage on delivery.
- ⚠ Never use a damaged product.
2. Check the product for damage and contamination before each use.
3. Use the product as intended. Remove the carrying strap before use!
4. Warm up before exercising.
5. Exercise according to the exercise instructions (see spinefitter.com) or as instructed by professionals.

Mounting on backrests

(See Page 38)

- ⚠ Risk of injury! The SPINEFITTER by SISSEL Mini is not suitable for use on car seat backrests.

1. Before use, check the backrest of your chair or seat to ensure that it is stable and can safely support the SPINEFITTER by SISSEL Mini.
2. Place the SPINEFITTER by SISSEL Mini on the backrest so that it is level with your lower to middle back and provides the desired support.
3. Attach the SPINEFITTER by SISSEL Mini firmly to the backrest using the adjustable carry strap. Make sure that it is secure and tight before using it.

4. The height of the SPINEFITTER by SISSEL Mini can be adjusted using the carry strap. Adjust it according to your personal preferences and needs to achieve maximum comfort and support.
5. Avoid excessive use of the SPINEFITTER by SISSEL Mini and take short breaks at regular intervals to prevent overstimulation. Sitting on one side for long periods of time can lead to tension and discomfort. Lean on the SPINEFITTER by SISSEL Mini for 6-8 minutes, then change the height of the SPINEFITTER by SISSEL Mini to reach other areas of the spine and gradually treat the thoracic and lumbar spine. If you only want to mobilize a partial area, pause after 6-8 minutes so as not to overstimulate the structures. When used in combination with the SPINEFITTER by SISSEL Trigger Tool, the application time is reduced to 30 seconds to a maximum of 2 minutes.
6. Use the SPINEFITTER by SISSEL Mini as a supplement to a healthy sitting posture, but also vary your sitting position and movements to reduce the strain on your back. Stand up occasionally, stretch or take a few steps to loosen your muscles and relieve tension.

Note: After use, cleaning and possible disinfection measures may be necessary (see Care & Hygiene).

5. CARE & HYGIENE

This product is suitable for reuse by additional persons. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed to ensure a safe operation. If required, clean the SPINEFITTER by SISSEL Multi Extension Kit with a soft cloth and mild soap and water solution. If required due to change of user, the surface of the SPINEFITTER by SISSEL Multi Extension Kit can be disinfected by wiping with an alcohol free solution. Disinfection products that contain caustic substances (e. g. acids, alkalis) should be avoided as they can damage the surface of the product.

- ⚠ Disinfection measures are to be carried out exclusively with suitable disinfectants that have been tested for effectiveness.
- ⚠ Disinfectants that can damage the materials of the product must be avoided.
- ⚠ The application instructions of the disinfectant manufacturer must be observed.

The use of unsuitable disinfection agents or agents that have not been tested for effectiveness or their incorrect use will invalidate the warranty and guarantee claims.

6. DISPOSAL & ENVIRONMENT

The product and the accessories supplied can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be observed.

7. WARRANTY

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL GmbH have been handled, treated or modified by others without consent or if the instructions for use have not been followed. Users of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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SPINEFITTER by SISSEL Multi Extension Kit: 1 x SPINEFITTER by SISSEL Mini, 1x Carry Strap, 1 x SPINEFITTER by SISSEL Connector, 1 x manuel d'instruction

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SISSEL GmbH · Bruchstrasse 48