# SPINEFITTER

BACK IN BALANCE

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BACK IN BALANCE

## 1. GENERAL INFORMATION

Read the following instructions for use carefully and observe the safety instructions and warnings.

The SPINEFITTER by SISSEL® is a product to support training, improve spinal and joint mobility, relieve tension and strengthen deep muscles.

As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them.

The therapeutic use of the SPINEFITTER by SISSEL®, e.g. in physiotherapy and rehabilitation treatment, should, if possible, be carried out by or under the guidance of professional users.

Independent use by non-professional users and medical laypersons is possible under consideration of the instructions for use. Initial instruction by appropriately trained professionals is recommended.

The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists.

SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these instructions for use.

The instructions for use must be kept carefully and within easy reach and must be enclosed if the product is passed on.

Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

## 2. SAFETY INFORMATION

- ! Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- ! Do not use the SPINEFITTER in the immediate vicinity of heat sources (e.g. radiators, hot-plates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the exercise area free from sharp and pointed objects.
- ! Exercise only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train exclusively barefoot or wear socks with a non-slip base. Do not train with shoes and make sure you wear suitable sports clothing.
- ! Choose a training programme that suits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.
- ! Do not insert fingers or other body parts into the openings of the training device.
- ! Do not bend the SPINEFITTER. Only use and store the SPINEFITTER flat and do not place any objects on it.
- ! Do not use the SPINEFITTER with more than one person at the same time.
- ! Only use accessories provided by the manufacturer.
- ! The carrying strap is used exclusively for transporting the SPINEFITTER and must be removed before use.
- ! The SPINEFITTER and its accessories are not toys and must be kept out of the reach of children. Do not allow children to play with it. Risk of injury and strangulation!

### 3. PRODUCT INFORMATION

#### Scope of delivery:

SPINEFITTER by SISSEL®: 1 x SPINEFITTER by SISSEL®, 1 x instruction manual, 1 x carrying strap

SPINEFITTER by SISSEL® Set of 10 incl. Coach Bag: 10 x SPINEFITTER by SISSEL®, 1 x Coach Bag, 1 x Instruction Manual

#### Technical specifications

##### Dimensions (L x W x H):

SPINEFITTER by SISSEL®	84 x 12 x 6 cm
Carrying strap	250 x 2 cm
Coach Bag	85 x 25 x 31 cm

##### Material

SPINEFITTER by SISSEL®	Polyurethane
Carrying Strap	Polyester
Coach Bag	Cotton

##### Max. user weight

SPINEFITTER by SISSEL®	150 kg
------------------------	--------

Note: The training device is made from a high-quality polyurethane foam. Please note that this naturally has a slightly structured, irregular surface. This does not represent a product defect and has no influence on the use or cleaning of the product.

#### Intended use

##### Use for general training

The SPINEFITTER by SISSEL® is an innovative training device that combines relaxation and effective training. For private or professionally guided individual or group training, the SPINEFITTER is suitable for a variety of exercises thanks to its unique shape and is an ideal support for releasing tension and adhesions and improving spinal and joint mobility.

By simply placing the body intuitively on the SPINEFITTER, a loosening, e.g. of the shoulder girdle, can take place and also stimulation of the energy points along the spine. This makes it the perfect partner for bringing the body into balance and promoting mobility.

### Therapeutic use

The SPINEFITTER by SISSEL® is used as a portable therapeutic aid for active loosening of myofascial tissue, relaxation of muscles close to the spine, individual massage, stimulation of blood circulation and metabolism as well as mobilisation of the spine in the area of the vertebral bodies. In addition, the product can be used as part of physiotherapeutic therapy programmes and as a therapeutic aid when standing for functional balance exercises, psychomotoric training, for training body awareness and for fall prevention exercises.

### **Indication**

- Chronic shoulder-neck tension
- Neck tension and neck pain
- Tense, hardened muscles near the spine
- Movement restrictions of the cervical spine, thoracic spine, shoulder joints
- Diffuse pain in the shoulder, neck, spine, lumbar spine
- Cervical, thoracic and lumbar spine syndrome
- Therapeutic aid for physiotherapy, osteopathy, sports medicine, rehabilitation
- Supporting aid for measures to improve posture, body awareness, body perception, movement sequences, balance, general coordination, psychomotoric skills, coordination
- Addition to and optimisation of the usual fitness programme

### **Contra indication**

- Lack of compliance (e.g. due to fear or pain)
- Acute injuries (open wounds, soft tissue injuries, unstable fractures in the spine, pelvis, extremities, etc.)
- Acute radiating pain, paraesthesia, movement disorders (consult specialist or physiotherapist)
- Osteoporosis
- Intervertebral disc problems
- Acute movement disorders
- Pronounced sensory perception disorders
- Usage by children and adolescents, during pregnancy or in case of general health problems only under professional supervision and after professional consultation

⚠ Risk of injury! Use the SPINEFITTER in case of pronounced sensory perception disorders, unstable fractures, open wounds, acute injuries and movement disorders only after prior consultation with a doctor.

⚠ Risk of injury! Only use the SPINEFITTER if you can ensure that the exercise recommendations described are carried out correctly. Supervision by qualified personnel is recommended.

### **Regulatory requirements**

The SPINEFITTER by SISSEL® GmbH fulfils the regulatory requirements of Regulation (EU) 2017/745 on medical devices.

⚠ All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

## Accessories

The SPINEFITTER may only be used with accessories offered or approved by the manufacturer (see [www.spinefitter.com](http://www.spinefitter.com)).

## 4. APPLICATION

1. Check the packaging and the product for any damage on delivery.

⚠ Never use a damaged product.

2. Remove the carrying strap before use.

3. Check the product for damage and contamination before each use.
4. Warm up before exercising.
5. Use the product as intended.
6. Exercise according to the exercise instructions (see [www.spinefitter.com](http://www.spinefitter.com)) or as instructed by professionals.

**Note:** After use, cleaning and possible disinfection measures may be necessary (see Care & Hygiene).

## 5. CARE & HYGIENE

- This product is suitable for reuse by additional persons. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed to ensure a safe operation.
- If required, clean the SPINEFITTER with a soft cloth and a mild soap and water solution.
- If required due to change of user, the surface of the SPINEFITTER can be disinfected by wiping with an alcohol free solution.
- Disinfection products that contain caustic substances (e. g. acids, alkalis) should be avoided as they can damage the surface of the product.

⚠ Disinfection measures are to be carried out exclusively with suitable disinfectants that have been tested for effectiveness.

⚠ Disinfectants that can damage the materials of the product must be avoided.

⚠ The application instructions of the disinfectant manufacturer must be observed.

- The use of unsuitable disinfection agents or agents that have not been tested for effectiveness or their incorrect use will invalidate the warranty and guarantee claims.

## 6. DISPOSAL & ENVIRONMENT

The product and the accessories supplied can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be observed.

## 7. WARRANTY

The warranty is governed by the statutory provisions.

Any warranty is excluded if goods delivered by SISSEL® GmbH have been handled, treated or modified by others without consent or if the instructions for use have not been followed.

Users of this product must observe the regulatory requirements for medical products applicable at the place of use.

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SISSEL® GmbH · Bruchstrasse 48